

AILE

Directions for Use



- **Product Name:** Reusable NIBP Cuff
- **Model:** CR009-R-PU-005
- **Applicability:** Adult, 9"~21" (22~53cm) arm circumference, single air hose, 0-300mmHg measurement range.
- **Intended use**
Reusable NIBP Cuff is an accessory used in conjunction with noninvasive blood pressure measurement systems. The cuff is non-sterile and may be reused. It is available in adult size.
- **Application method**
 - (1) Wrap the cuff around the patient's upper arm, with the side marked "THIS SIDE TO PATIENT" facing patient's skin. The ARTERY arrow points in the direction of the wrist.
 - (2) Adjust the cuff position to locate the ARTERY arrow 2-3cm above the arm fossa, and make sure the INDEX LINE is between in the RANGE lines.
 - (3) Connect the air hose to non-invasive blood pressure measurement device, and adjust the posture and prepare to start the measurement according to non-invasive blood pressure measuring instructions.
- **Cleaning and Maintenance**
 - (1) Wet a clean cloth with 70% alcohol, and no liquid drips.
 - (2) Wipe the cuff and air hose carefully, until no dirt or stains can be seen.
 - (3) Dry the cuff naturally, and then fold it up for next use.

- **Storage/Transportation**

Temperature (Operating): +5°C to +40°C

Temperature (Storage / Transportation): -10°C to +40°C

Humidity (Operating / Storage / Transportation): 0~80% non-condensing

Hyperbaric Pressure (Operating / Storage / Transportation): 86kpa~106kPa

During transportation, collision, violent vibration and direct rain and snow should be avoided.

- **Warnings**

- (1) Do not use the product for unintended use.
- (2) Check the product integrity before use.
- (3) Replace the product with a new one if there is any cracks or holes.
- (4) Discard the product as medical waste according to region laws, to avoid potential cross contamination.

- **Warranty**

After-sales information:

5 year warranty. If you have any problems.

Please contact the following for warranty:

Web: <https://4aile.com>

E-mail: support@4aile.com

Phone: +1 888-875-0846

The following symbols may appear on the product or product labeling:

See instructions before use	Manufacturer	Non sterile packaging	Latex Free	Do not discard carelessly
Lot Number	Expiration Date	Date of manufacture	Caution: Federal (USA) law restricts this device to sale by or on order of a physician	



Shenzhen Coreray Technology Co., Ltd

Add: Floor 5, Building 10, Huangbeiling Jingxuan Industrial Park, Yousong Community, Longhua District, Shenzhen, Guangdong, China

TEL: +86-755-21010817